

# GoCHx™ GEL

## 0.8% Syringable Chlorhexidine Gel

### DESCRIPTION AND INTENDED USE:

GoCHx™ Gel is a syringeable thin, non-alcohol based gel containing 0.8% Chlorhexidine in a water soluble formula. Because of its water solubility, and low viscosity, when applied to dental restorations, it acts as a cleansing agent and does not affect bond strengths. This gel is provided in easy to use syringes with flock tips to precisely deliver the gel to the intended areas. External cleaning solution.

GoCHx™ Gel is recommended for use on dental restorations, prior to insertion. After applying GoCHx™ Gel to cleanse the restoration, the surface of the restoration should be rinsed thoroughly with water, and then air dried prior to insertion. For use only by a licensed dentist. Rx use only. Non-sterile. No clinical testing has been performed to demonstrate antibacterial effectiveness.

### Directions for Use:

- For new restorations received from dental lab: Follow steps 3-8.
  - For restorations in patients mouth: Follow steps 2-8.
- Remove crown, bridge, or temporary restoration from the mouth.
- Wash surface of the restoration to remove any debris.
- Place flock tip applicator on the GoCHx™ Gel syringe.
- Apply GoCHx™ Gel onto the surface of the restoration with gentle pressure on the syringe barrel, while gently scrubbing the restoration surface with the flock tip.
- Completely wash off GoCHx™ Gel and air dry the restoration surface. Restoration is now ready for seating.
- For Zirconia Surfaces ONLY (ie. Restorations and abutments.) It is necessary to restore phosphate groups after try-in using a zirconia primer to restore bond strengths zirconia.
- Seat and finalize the restoration with doctor's preferred method (cementation or screw retained).
- Repeat steps 3-8 for additional restorations.

### Storage Conditions, Precautions, and Disposal:

**Storage:** Store at room temperature. Individuals known to be sensitive to Chlorhexidine should avoid product use. **Precautions: If accidental exposure occurs: Eyes** – Wash eyes at an eye washing station. **Skin** – Wash skin with soap and warm water. **Ingested** – If concerned seek physician guidance. **If irritation persists** after accidental exposure occurs, discontinue use and seek medical care. **Handling:** Dental personnel are advised to wear surgical gloves and appropriate PPE (personal protective equipment). **Disposal:** Dispose of properly following all local, state, and federal government guidelines. Please see SDS for more information.



277 NEW YORK AVENUE, JERSEY CITY, NJ 07307

TELE: 800.828.2634 / 201.798.5353 FAX: 201.659.7186

EMAIL: SALES@TAUBDENTAL.COM

WEBSITE: HTTP://WWW.TAUBDENTAL.COM