



DESCRIPTION AND INTENDED USE:

Liquid Magic™ is a light-cured, resin barrier for Implant and Cosmetic Dentistry. It is recommended for protecting the screw head and sealing the screw access hole on abutments and screw retained restorations from cements and composite restoratives. Use on screw threads and internal implant components to offering temporary protection during recall and debriding. Liquid Magic™ when cured is a flexible, soft material allowing for easy removal when desired, often in a single piece, making clean-up fast and complete.

SAFETY PRECAUTIONS:

Liquid Magic™ contains acrylate resins. Some individuals may be sensitive to such resins. Those individuals known to be sensitive to acrylate resins should avoid use of this product. Dental personnel should wear appropriate gloves and clothing (PPE) to protect them from unnecessary exposure.

DIRECTIONS FOR USE:

1. Liquid Magic™ should be placed on clean surfaces which are not excessively wet or contaminated with saliva or other debris. A fresh applicator tip should be used for each new application. Between uses, replace the screw cap on the syringe barrel.
2. To apply Liquid Magic™ gentle pressure should be placed on the syringe plunger. Liquid Magic™ is a thin resin which will settle to fill crevices and low lying areas. Do not over fill areas of application.
3. After desired placement is achieved, light-cure Liquid Magic™ for 20 seconds for each area which the curing light can cover. Additional curing time may be required for thick applications of material exceeding 5 mm. Strong curing units may require less curing time.
4. Liquid Magic™ when cured is a flexible, soft material allowing for easy removal when desired, often in a single piece, making clean-up fast and complete. Dispose of the removed material in a safe storage container, particularly if the removed material has been in contact with biological fluids.
5. After use, replace the cap on the syringe and safely dispose of the applicator tip. Store Liquid Magic™ at room temperature.

For Use only by a Licensed Dentist. Rx Use Only.



TAUB
PRODUCTS

800.828.2634
SALES@TAUBDENTAL.COM
WWW.TAUBDENTAL.COM